



Over 30 years of partnership with the
global foot & ankle community




Orthopaedic Devices, Foot & Ankle Products

Preventive, Postoperative, Off-loading, Wound Care



DARCO

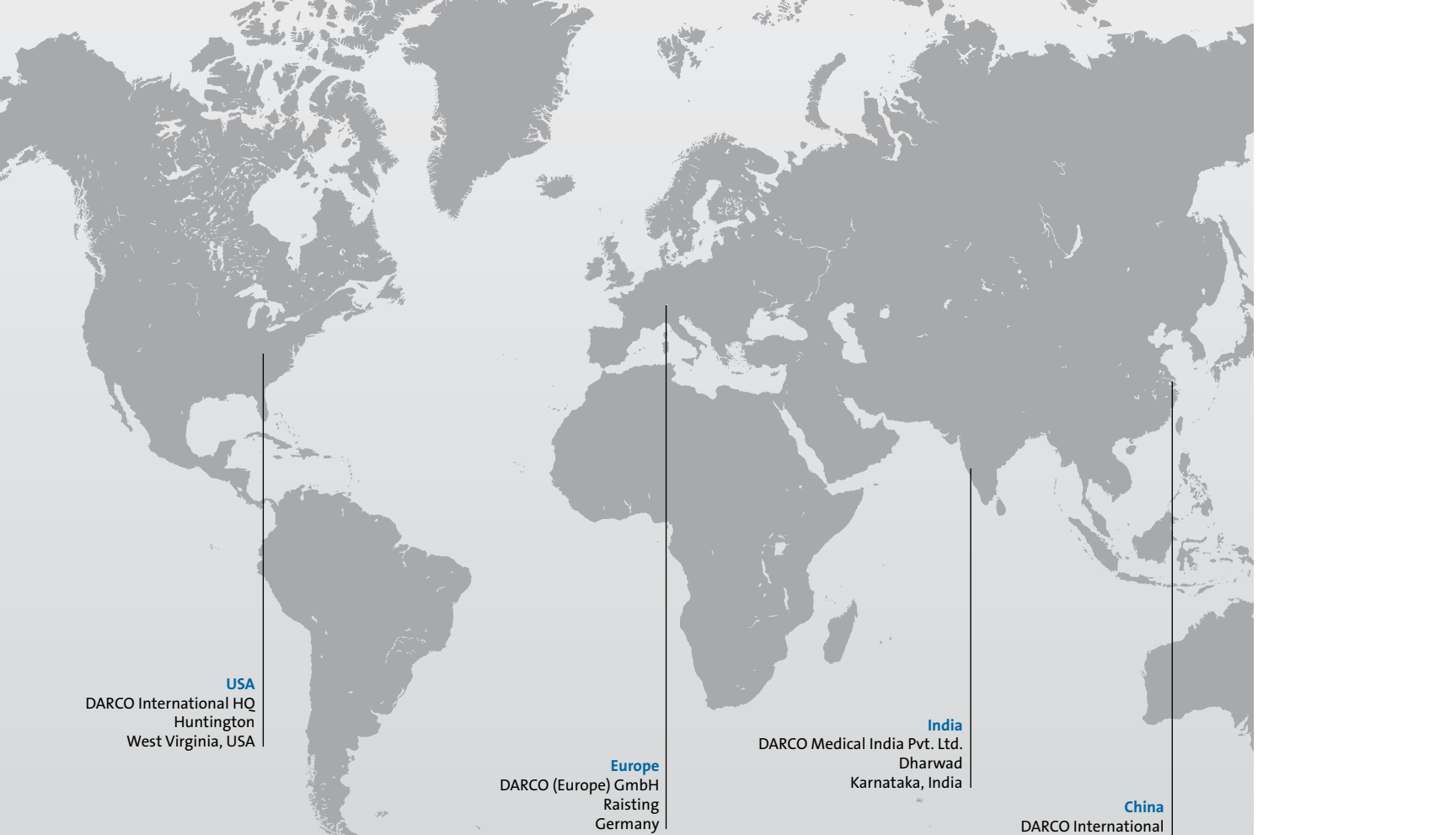

USA
DARCO International HQ
Huntington
West Virginia, USA

Europe
DARCO (Europe) GmbH
Raisting
Germany

India
DARCO Medical India Pvt. Ltd.
Dharwad
Karnataka, India

China
DARCO International
Trading Shanghai Ltd.
China

DARCO International Locations

Celebrating over 30 years as a global leader in the design, manufacture and distribution of innovative products for the foot and ankle community.

DARCO International is dedicated to being the leading provider of post-op, trauma and wound care solutions to the global foot and ankle community.

Beyond the manufacture and sale of its products, DARCO prides itself on being a good citizen in the market place, the profession and the community. Philanthropy is a part of the DARCO credo with significant ongoing contributions to medical education, with emphasis on the improvement of the medical and surgical treatment of lower extremity problems.

DARCO is committed to the promotion and education of physicians engaged in the medical

and surgical treatment of the foot and ankle throughout the world. DARCO-sponsored symposiums and workshops have been successfully held in Germany, Austria and China.

DARCO has contributed over \$500,000 to the Fund for Podiatric Medical Education (FPME) to provide scholarships to podiatry students nationwide and continues to be an active supporter of both our schools and our students.

Table of Contents



Postoperative Care

- | | |
|--|--|
| <ul style="list-style-type: none"> 7 MedSurg™
Original MedSurg™ 8 Softie™ Shoe
APB™ - All Purpose Boot 9 HeelWedge™
OrthoWedge™ | <ul style="list-style-type: none"> 10 GTS™ - Great Toe Splint
TAS™ - Toe Alignment Splint 11 Body Armor® Toe Guard |
|--|--|



Trauma Care / Walkers / Cast Shoes

- | | |
|---|--|
| <ul style="list-style-type: none"> 13 Body Armor® Cast Shoe
CBO™ 14 SlimLine™
Air Traveler™ | <ul style="list-style-type: none"> 15 FX Pro™
Body Armor® Walker 16 Body Armor® Walker II
Body Armor® Pro Term |
|---|--|



Off-loading and Wound Care

- | | |
|--|--|
| <ul style="list-style-type: none"> 19 PegAssist™ Insoles (PTQ, PQ, & PW Series) 20 WCS™ Open Toe
WCS™ Closed Toe | <ul style="list-style-type: none"> 21 HeelWedge™
OrthoWedge™ 22 MedSurg™ with PegAssist (PTQ Series) 23 GentleStep™ |
|--|--|



Plantar Fasciitis Products

- 25 Pneumatic Night Splint
Body Armor® Night Splint
- 26 DCS™ - Plantar Fasciitis Sleeve

Ankle Supports

- 29 Body Armor® Vario
Body Armor® Sport
- 30 Web™ Ankle Brace



Kinesiology Tape

- 33 MECRON Elastic Tape



Orthotic Insoles

- 35 ProMotion® Plus



Pediatric Products

- 37 Pediatric MedSurg™
Pediatric Original MedSurg™
- 38 Pediatric SlimLine™
Pediatric CBO™

Diagnostic Products

- 39 FootFilament™
Monofilament Tester

Additional Information

- | | |
|--|--|
| <ul style="list-style-type: none"> 40 Insole Size Chart | <ul style="list-style-type: none"> 42 Diabetic Treatment Product Guide 43 Product Videos |
|--|--|

Product Quick Guide

		Postoperative Healing	Trauma Care	Fractures	Sprains	Weak Ankles	Plantar Fasciitis	Wounds/Off-loading	Presence of K-Wires	Cast Protection	Diabetic Foot/Neuropathy	All Weather Protection	Suggested HCPCS Code	
Air Traveler™ Page 14		✓	✓	✓	✓	✓								L4360
APB™ All Purpose Boot Page 8		✓	✓						✓	✓	✓	✓	✓	L3260
Body Armor® Cast Shoe Page 13				✓							✓	✓	✓	L3260
Body Armor® Night Splint Page 25								✓						L4396
Body Armor® Pro Term Page 16		✓	✓											L4360 L4361
Body Armor® Sport Page 29					✓	✓								L1902
Body Armor® Toe Guard Page 11		✓	✓							✓				L9900
Body Armor® Vario Page 29		✓	✓		✓	✓							✓	L1906
Body Armor® Walker Page 15		✓	✓	✓	✓	✓								L4386
Body Armor® Walker II Page 16		✓	✓	✓	✓	✓		✓						L4386
CBO™ Page 13, 38				✓							✓	✓		L3260
MECRON Elastic Tape Page 33					✓	✓	✓							
DCS™ - Plantar Fasciitis Sleeve Page 26								✓					✓	
FX Pro™ Page 15		✓	✓	✓	✓	✓								L4386
GentleStep™ Page 23												✓		A5500 L3216 L3221
GTS™ - Great Toe Splint Page 10		✓												L3100
HeelWedge™ Page 9, 21		✓						✓						L3260
MedSurg™ Page 7, 22, 37		✓	✓						✓	✓		✓		L3260
Night Splint Wedge Page 26								✓						L4396


		Postoperative Healing	Trauma Care	Fractures	Sprains	Weak Ankles	Plantar Fasciitis	Wounds/Off-loading	Presence of K-Wires	Cast Protection	Diabetic Foot/Neuropathy	All Weather Protection	Suggested HCPCS Code	
Original MedSurg™ Page 7, 37		✓	✓							✓	✓			L3260
OrthoWedge™ Page 9, 21		✓	✓						✓	✓				L3260
PTQ Series PegAssist™ Page 19		✓	✓						✓					A9283
PQ Series PegAssist™ Page 19		✓	✓						✓					A9283
PW Series PegAssist™ Page 19		✓	✓						✓					A9283
Pneumatic Night Splint Page 25								✓						L4396
ProMotion® Plus Page 35								✓					✓	L3040
SlimLine™ Cast Boot Page 14, 38			✓	✓						✓	✓	✓		L3260
Softie™ Shoe Page 8		✓	✓							✓	✓			L3260
TAS™ - Toe Alignment Splint Page 10		✓												L3100
WCS™ - Open-Toe Page 20			✓						✓			✓		A9270
WCS™ - Closed-Toe Page 20			✓						✓			✓		A9270
Web™ Ankle Brace Page 30						✓	✓							L1902



Postoperative Care

MedSurg™

Postoperative Shoe

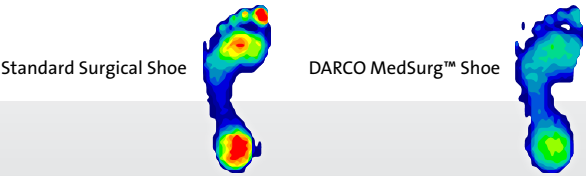


The MedSurg™ Shoe is one of the most versatile postoperative shoes on the market today. EVA insole features twice the padding of standard insoles and can be modified as needed or removed to accommodate the PegAssist™ off-loading insole.

- › Square-toe design acts as a bumper to protect the toes when k-wires are present and provides better left/right fit.
- › MetaShank™ protection provides more rigid control under the metatarsal heads.
- › Forefoot closure eliminates buckle pressure on the forefoot and can expand to accommodate dressing up to 17 inches in circumference.

Suggested Code: L3260

Compatible with:

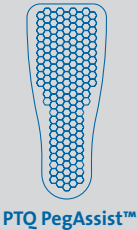


MedSurg™

Size:	Women:	Men:	Product Codes	
			Women:	Men:
S	4 - 6	6 - 8	MQW1B	MQM1B
M	6.5 - 8	8.5 - 10	MQW2B	MQM2B
L	8.5 - 10	10.5 - 12	MQW3B	MQM3B
XL	-	12.5 - 14	-	MQM4B

Original MedSurg™

Postoperative Shoe

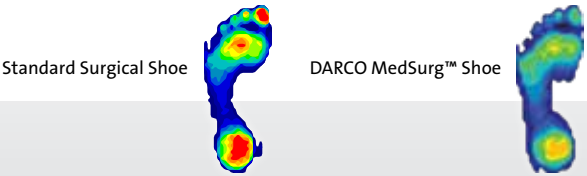

The DARCO medical-surgical shoe is the shoe that started it all. Our original MedSurg™ shoe was invented over 30 years ago and has since become known as the industry standard. Often referred to generically as “the DARCO shoe”, it is anything but generic.

- › MetaShank™ provides superior control of forefoot motion to ensure postoperative healing remains undisturbed. Better protection after osteotomies.
- › Reinforced padded heel grips the foot and reduces slippage.
- › Breathable mesh upper light weight material allows for proper ventilation and conforms to bony abnormalities.
- › Extra long straps accommodate even the bulkiest dressings.

Suggested Code: L3260

Compatible with:



Original MedSurg™

Size:	Women:	Men:	Product Codes	
			Women:	Men:
S	4 - 6	6 - 8	MSW1N	MSM1N
M	6.5 - 8	8.5 - 10	MSW2N	MSM2N
L	8.5 - 10	10.5 - 12	MSW3N	MSM3N
XL	-	12.5 - 14	-	MSM4N

Softie™ Shoe

Postoperative Shoe



The Softie™ Shoe is a cost-effective alternative to the more premium shoes while still providing the rigidity and control of the forefoot you have come to expect from all DARCO products.

- › Semi-rigid metatarsal shank provides postoperative healing and better protection after osteotomies.
- › Tri-laminated upper made of light weight tricot-foam material is breathable and comfortably conforms to bony abnormalities.
- › The hook and loop straps are easy to fasten and unfasten with one hand.

Suggested Code: L3260

Compatible with:




Body Armor® Toe Guard - Round-toe

Softie™ Shoe

Size:	Women:	Men:	Product Codes:	
			Women:	Men:
S	4 - 6	6 - 8	STW1B	STM1B
M	6.5 - 8	8.5 - 10	STW2B	STM2B
L	8.5 - 10	10.5 - 12	STW3B	STM3B
XL	-	12.5 - 14	-	STM4B

APB™

All Purpose Boot

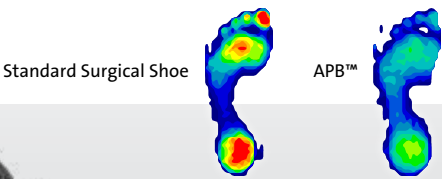


The APB™ embodies nearly perfect four season postoperative protection with a closed-toe design that keeps dressings and toes warm and dry.

- › Relaxed rocker sole reduces plantar pressure by over 25%.
- › Square-toe design acts as a bumper to protect the toes.
- › Ankle strap seats foot firmly in shoe.
- › Removable forefoot closure eliminates buckle pressure.
- › Zoned Outsole has an aggressive tread under the mid-foot.
- › MetaShank™ under the MP joints increases support and rigidity.
- › Insole features twice the padding of standard insoles and can be modified as needed.

Suggested Code: L3260

Compatible with:

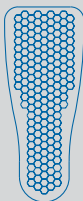

PQ PegAssist™


APB™ - All Purpose Boot

Size:	Women:	Men:	Product Codes:
XS	4 - 7	-	APQ0B
S	7.5 - 10	6 - 8	APQ1B
M	10.5 - 13	8.5 - 10	APQ2B
L	13.5+	10.5 - 12	APQ3B
XL	-	12.5 - 14	APQ4B

HeelWedge™

Off-loading Shoe

Standard Surgical Shoe

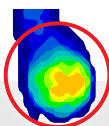

DARCO HeelWedge™




The HeelWedge™ is clinically proven to off-load pressure from the heel by shifting weight to the mid and forefoot to promote faster healing when wounds or ulcerations are present on the heel.

- › Reduces heel pressure by over 26%!
- › Square-toe design acts as a bumper to protect the toes.
- › Ankle strap seats foot firmly in shoe.
- › Removable forefoot closure eliminates buckle pressure.
- › Zoned outsole has an aggressive tread under the midfoot where traction is needed most.
- › EVA insole features twice the padding of standard insoles and can be modified.

Suggested Code: L3260

Compatible with:

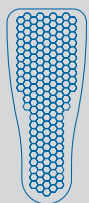

PQ PegAssist™

HeelWedge™

Size:	Women:	Men:	Product Codes:
XS	4 - 7	-	HQ0B
S	7.5 - 10	6 - 8	HQ1B
M	10.5 - 13	8.5 - 10	HQ2B
L	13.5+	10.5 - 12	HQ3B
XL	-	12.5 - 14	HQ4B



Video demonstration:

How to walk properly while wearing the HeelWedge™


Scan QR code or visit:

darcointernational.com/heelwedge

OrthoWedge™

Off-loading Shoe

Standard Surgical Shoe

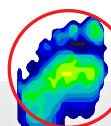

DARCO OrthoWedge™




The OrthoWedge™ is clinically proven to reduce weight bearing pressure on the forefoot which promotes faster healing when forefoot wounds or ulcerations are present.

- › Reduces forefoot pressure by as much as 57%!
- › Square-toe design acts as a bumper to protect the toes.
- › Ankle strap seats foot firmly in shoe to reduce heel slippage and eliminate friction.
- › Removable forefoot closure eliminates buckle pressure.
- › Zoned outsole has an aggressive tread under the midfoot.
- › EVA insole features twice the padding of standard insoles and can be modified as needed.

Suggested Code: L3260

Compatible with:

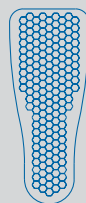

PQ PegAssist™



Body Armor® Toe Guard -
Square-toe

OrthoWedge™

Size:	Women:	Men:	Product Codes:
XS	4 - 7	-	OQ0B
S	7.5 - 10	6 - 8	OQ1B
M	10.5 - 13	8.5 - 10	OQ2B
L	13.5+	10.5 - 12	OQ3B
XL	-	12.5 - 14	OQ4B



Video demonstration:

How to walk properly while wearing the OrthoWedge™

Scan QR code or visit:

darcointernational.com/orthowedge

GTS™

Great Toe Splint



The GTS™ - Great Toe Splint is an adjustable, postoperative splint. Indicated for relief of symptoms associated with hallux valgus and other great toe joint conditions. Manufactured with lightweight neoprene and nylon, the GTS™ can be worn day and night, and its form-fitting design easily accommodates suitable footwear.

- › Maintains alignment
- › 24 hour support
- › Helps relieve painful bunions
- › Can be worn postoperatively

Suggested Code: L3100



GTS™ - Great Toe Splint

Size:	Avg. Shoe Size	Product Codes	
		Right:	Left:
S	W 5-7.5 / M 3.5-6	GTS1R	GTS1L
L	W 8-11 / M 6.5-9.5	GTS2R	GTS2L

TAS™

Toe Alignment Splint



The TAS™ - Toe Alignment Splint is a cost-effective solution that provides superior postoperative toe alignment after Hallux Valgus, Hammer Toe and Tailor's Bunion procedures.

- › Low profile metatarsal band fits easily into most footwear and reduces slippage
- › Special t-strap gently maintains proper alignment for post-procedural healing
- › One size fits most
- › Latex free

Suggested Code: L3100



TAS™ - Toe Alignment Splint

Size:	Product Code:
Universal	TAS



Video demonstration:
How to properly apply the TAS™

Scan QR code or visit:
darcointernational.com/tas

Body Armor® Toe Guard

Toe Protection



The Body Armor® Toe Guard provides extra protection for the toe region during the recovery process. It works particularly well when k-wires are present or anytime when accidental contact could cause pain or be detrimental to the healing process.

- › Hard ABS construction is durable and withstands blows and strikes to the toe region in everyday situations.
- › Moldable side wings can be bent or molded with heat gun to conform to any foot size.
- › Strong adhesive backing ensures secure, lasting fit.
- › Round-toe or square-toe versions available.

Suggested Code: L9900

Compatible with:



MedSurg™



Softie™




MedSurg™ Original



SlimLine™



OrthoWedge™

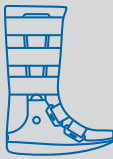

FX Pro™



Air Traveler™

Body Armor® Toe Guard

Size:	Square - toe:	Round- toe:
WS - WL	BATGQ1	BATGS1
MS - MXL	BATGQ2	BATGS2





Trauma Care, Walkers, Cast Shoes

Body Armor® Cast Shoe

Cast Boot



The Body Armor® Cast Shoe is specifically engineered to shield the cast from moisture, shock and everyday wear. The shoe can be used with fiberglass casts and bulky bandages.


- › The new design features sturdier low profile buttons, a more secure locking mechanism, a stronger bungee system, and a redesigned body that better adapts to the foot.
- › EVA Unibody Design provides shock absorption and helps protect the cast.
- › Unique Bungee Closure System is easy to use, easy to adjust and effectively secures the cast in the shoe.
- › Built in Toe Guard helps shield the toes.
- › Style that sets it apart; Function that puts it in a class of its own.

Suggested Code: L3260



Body Armor® Cast Shoe Sizing in Inches:

Size:	Length (A):	Width (B):	Width (C):	Product Codes:
XS	9.4"	3.9"	3.5"	BACSO
S	10"	4.4"	4.1"	BACS1
M	10.6"	4.7"	4.3"	BACS2
L	11.4"	5.1"	4.6"	BACS3
XL	12.6"	5.5"	4.8"	BACS4
2XL	13.7"	5.9"	5.1"	BACS5

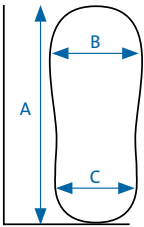

CBO™

Open Toe Cast Boot



The Darco CBO™ is a low profile multi use cast boot intended to protect casts of all types and heavy compression bandages.

- › Heavy duty canvas upper that protects the cast from the elements.
- › Multi layer EVA sole provides extra cushioning for ease of ambulation and durability.
- › The rocker sole helps ease pressure when ambulating and allows an easy roll through the gait pattern.
- › Durable hook and loop fastener straps seat and secure the casted foot and keep the boot snug around the cast/bandaging.
- › The CBO™ is lightweight and low profile.

Suggested Code: L3260



CBO™

Size:	Women:	Men:	Product Codes:
XS	< 5	-	CBO0
S	5.5 - 7.5	-	CBO1
M	8 - 10	6 - 8.5	CBO2
L	-	9 - 11	CBO3
XL	-	11.5 - 14	CBO4

SlimLine™

Cast Boot




The SlimLine™ Cast Boot can be adapted to fiberglass casts, bulky compression bandages and a variety of wound care applications, making it one of the most versatile cast shoes on the market.

- › Square-toe design acts as a protective bumper and gives the patient more room to accommodate larger casts and bandages.
- › Universal forefoot closure.
- › Higher ankle strap adds stability.
- › Elasticized strap allows for fixed ankle ambulation.
- › Relaxed rocker sole allows for a smooth roll throughout the gait pattern.

Suggested Code: L3260

Compatible with:

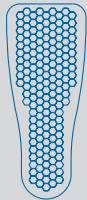

PQ PegAssist™



Body Armor® Toe Guard - Square-toe

SlimLine™ Cast Shoe

Size:	Women:	Men:	Product Codes:
XS	< 5	-	SLQ0B
S	5.5 - 8	-	SLQ1B
M	8.5 - 10	6 - 8	SLQ2B
L	10.5 - 13	8.5 - 11	SLQ3B
XL	13.5+	11.5 - 14	SLQ4B

Air Traveler™

Pneumatic Walker




The Air Traveler™ walker features a circumferential air bladder that provides the ultimate compression and security for the wearer. The inflation/deflation system allows the patient to easily adjust pressure, and the five-way strapping system adjusts to fit virtually any leg.

- › Circumferential air bladder provides custom fit and unparalleled compression.
- › Low profile sole more closely aligns with shoe height to allow for a more natural stride.
- › 10mm EVA insole gives the patient the support they need with the comfort they desire.

Suggested Code: L4360

Compatible with:

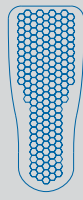

PW PegAssist™



Body Armor® Toe Guard - Square-toe

Air Traveler™

High Version:	Women:	Men:	Product Codes:
XS	3.5 - 6	-	AT0
S	6.5 - 9	5 - 7.5	AT1
M	9.5 - 12	8 - 10.5	AT2
L	12.5+	11 - 13	AT3
XL	-	13.5+	AT4

Low Version:	Women:	Men:	Product Codes:
XS	3.5 - 6	-	ATS0
S	6.5 - 9	5 - 7.5	ATS1
M	9.5 - 12	8 - 10.5	ATS2
L	12.5+	11 - 13	ATS3
XL	-	13.5+	ATS4

FX Pro™

Walker




The re-designed FX Pro™ Walker features a more rigid shell with a soft direct injection outsole for better traction and less pressure at heel strike. Our low profile design eliminates leg length discrepancies associated with other walkers and is built with a wider platform to better accommodate edema or large bandages. The new “soft touch” band edging the stirrups and softer strapping materials provides greater patient comfort and compliance.

- › Low Profile design.
- › Padded liner is ventilated which promotes air circulation.
- › Anatomically contoured “spatula” stirrups immobilize the ankle.
- › Pressure moldable, shock absorbing insole conforms to the foot for maximum comfort.

Suggested Code: L4386

Compatible with:

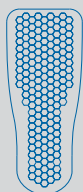

PW PegAssist™



Body Armor® Toe Guard - Square-toe

FX Pro™

High Version:	Women:	Men:	Product Codes:
XS	3.5 - 6	-	FX0
S	6.5 - 9	5 - 7.5	FX1
M	9.5 - 12	8 - 10.5	FX2
L	12.5+	11 - 13	FX3
XL	-	13.5+	FX4

Low Version:	Women:	Men:	Product Codes:
XS	3.5 - 6	-	FXS0
S	6.5 - 9	5 - 7.5	FXS1
M	9.5 - 12	8 - 10.5	FXS2
L	12.5+	11 - 13	FXS3
XL	-	13.5+	FXS4

Body Armor® Walker

Cast Replacement



DARCO's original rigid walker provides unparalleled protection, compression and immobilization for foot and ankle conditions where ambulation is permitted. It is the perfect alternative to a weight-bearing cast for compliant patients.

- › Rigid shell provides uniform compression which aids to reduce edema and secure immobilization.
- › Ski-boot closure allows every user to find a secure and comfortable fit.
- › Removable liner can be laundered and replaced.

Suggested Code: L4386



Body Armor® Walker

High Version:	Women:	Men:	Product Codes:
S	7 - 8.5	6 - 7.5	BAW1
M	9 - 11.5	8 - 10.5	BAW2
L	12+	11 - 14	BAW3

Low Version:	Women:	Men:	Product Codes:
S	7 - 8.5	6 - 7.5	BAWS1
M	9 - 11.5	8 - 10.5	BAWS2
L	12+	11 - 14	BAWS3

Body Armor® Walker II

Cast Replacement



The Body Armor® Walker II features a built-in liner and raised air vents for cooling. It comes with a removable Plastazote® insole that can be modified to off-load the plantar surface.

- › Rigid shell provides uniform compression which helps to reduce adema and secure immobilization.
- › Ski-boot closure allows every user to find a secure and comfortable fit.
- › The upper conforms to the leg and provides more uniform compression of the foot and ankle.

Suggested Code: L4386



Body Armor® Walker II

High Version:	Women:	Men:	Product Codes:
XS	< 4	-	BAT0
S	4.5 - 6.5	< 6	BAT1
M	7 - 10	6.5 - 9	BAT2
L	10.5 - 13	9.5 - 13	BAT3
XL	-	> 13.5	BAT4
Low Version:	Women:	Men:	Product Codes:
XS	< 4	-	BATS0
S	4.5 - 6.5	< 6	BATS1
M	7 - 10	6.5 - 9	BATS2
L	10.5 - 13	9.5 - 13	BATS3
XL	-	> 13.5	BATS4

Body Armor® Pro Term

Foot Stump Orthosis



The Body Armor® Pro Term is an interim orthosis providing a conservative follow-up treatment for Chopart and Lisfranc amputations. The foot stump orthosis may be worn for approximately 10 weeks following amputation. The Pro Term allows the foot stump to heal prior to transitioning to a custom made shoe.

- › Wide opening for easy foot insertion.
- › Practical & secure closure system.
- › Air compression bladder for customizable fitting.
- › Can be worn on the left or right.

Suggested Codes: L4360, L4361



Body Armor® Pro Term

Size:	Stump Length:	Product Codes:
M	6.3 in	BAPT2
L	7.5 in	BAPT3






Off-loading and Wound Care

PegAssist™

PQ, PTQ, & PW Series Off-loading Insoles

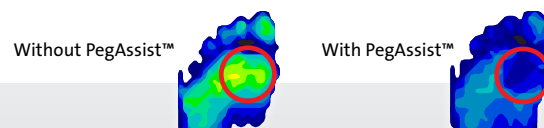

PTQ Series Off-loading Insole



PQ Series Off-loading Insole



PW Series Off-loading Insole

PTQ Series Insole



MedSurg™

The PTQ Series PegAssist™ Insole is designed to fit the DARCO Square-Toe MedSurg™ Shoe creating a cost-effective option for wound care and one that significantly reduces pressure postoperatively.

PQ Series Insole



HeelWedge™



OrthoWedge™



APB™



SlimLine™

The PQ Series PegAssist™ Insole is built around a multi-purpose removable peg insole designed to fit the HeelWedge™, OrthoWedge™, APB™ and SlimLine™.

PW Series Insole



Walkers

The PW Series PegAssist™ is an easy to use off the shelf product that quickly turns most standard walkers into an effective off-loading device. Affixed to the stabilizer board is a low profile molded hook strip that adheres to the walker liner to prevent shifting while walking.

The DARCO PegAssist™ Off-loading Insoles features a removable-peg chassis that effectively off-loads the plantar aspect of the foot after surgery or when wounds are present.

- › Removable pegs allow for localized off-loading of wounds and ulcerations of the foot.
- › Pressure is reduced by as much as 60% while allowing patient to remain ambulatory.
- › Each insole has a 15mm combination Plastazote® and Multi-Foam base with a 3mm PORON® Cushioning cover.
- › PORON® Cushioning cover eliminates the incidence of ring edema and edge abrasion.
- › Stabilizer board included to maintain product integrity.

Suggested Code: A9283

3mm PORON®
Cushioning Cover

Plastazote®

Removable EVA
Plantar Pegs

Stabilizer Board

PTQ Series PegAssist™

Size:	Product Code	
	Women:	Men:
S	4 - 6	6 - 8
M	6.5 - 8	8.5 - 10
L	8.5 - 10	10.5 - 12
XL	-	12.5 - 14

PQ Series PegAssist™

Size:	Product Code	
	Women:	Men:
XS	4 - 7	-
S	7.5 - 10	6 - 8
M	10.5 - 13	8.5 - 10
L	13.5+	10.5 - 12
XL	-	12.5 - 14

PW Series PegAssist™

Size:	Product Codes:	
	Women:	Men:
XS	3.5 - 6	-
S	6.5 - 9	5 - 7.5
M	9.5 - 12	8 - 10.5
L	12.5+	11 - 13
XL	-	13.5+



Video demonstration:

Proper assembly and application of the PegAssist™ Off-loading Insole

Scan QR code or visit:

darcointernational.com/pegassist

WCS™ Open-Toe

Wound Care Shoe System

WCS™ Closed-Toe

Wound Care Shoe System



The WCS™ Wound Care Shoe System is the product of choice when dealing with lesions through Wagner Grade 3 ulcerations. The four multi-density insoles can be modified, effectively decreasing pressure from the plantar aspect of the foot.

- › Four multi-density insoles allow for the ultimate in customization and precisely targeted off-loading.
- › Ultra-soft Plastazote® lining provides a virtually friction-free contact surface that won't exacerbate existing wounds. (Open-Toe only)
- › Genuine leather upper can be modified or cut away to provide even greater levels of pressure relief. (Open-Toe only)
- › Sold in pairs to eliminate discrepancies in shoe height.

Indications

- › Up to Wagner Grade 3 ulcerations
- › Bony prominences
- › Long-term treatment of wounds and ulcerations
- › Maximum off-loading while still allowing for patient ambulation



Video demonstration:
How to properly modify and apply the WCS™

Scan QR code or visit:
darcointernational.com/wound-care-shoe-system

Suggested Code: A9270

WCS™ Cross Section



- ← Plastazote® Thickness 1/4" Durometer 26
- ← Medium Density EVA Thickness 1/4" Durometer 42
- ← Firm Density EVA Thickness 1/4" Durometer 50
- ← PORON® Cushioning Thickness 1/8" Durometer 29



Replacement insoles available.
Sold separately.



WCS™ Open-Toe

Size:	Women:	Men:	Product Codes:	Replacement Insoles:
XS	4 - 5.5	-	WCS0B	WCI0
S	6 - 8.5	6 - 7.5	WCS1B	WCI1
S/M	9 - 9.5	8 - 8.5	WCS7B	WCI7
M	10 - 10.5	9 - 9.5	WCS2B	WCI2
L	11 - 12.5	10 - 11.5	WCS3B	WCI3
XL	13 - 14.5	12 - 13.5	WCS4B	WCI4
2XL	15 - 17	14 - 16	WCS5B	WCI5


WCS™ Closed-Toe

Size:	Women:	Men:	Product Codes:	Replacement Insoles:
XS	4 - 5.5	-	WCC0B	WCI0
S	6 - 8.5	6 - 7.5	WCC1B	WCI1
S/M	9 - 9.5	8 - 8.5	WCC7B	WCI7
M	10 - 10.5	9 - 9.5	WCC2B	WCI2
L	11 - 12.5	10 - 11.5	WCC3B	WCI3
XL	13 - 14.5	12 - 13.5	WCC4B	WCI4
2XL	15 - 17	14 - 16	WCC5B	WCI5


HeelWedge™

Off-loading Shoe

Standard Surgical Shoe

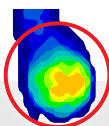

DARCO HeelWedge™

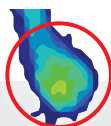


The HeelWedge™ is clinically proven to off-load pressure from the heel by shifting weight to the mid and forefoot to promote faster healing when wounds or ulcerations are present on the heel.

- › Reduces heel pressure by over 26%!
- › Square-toe design acts as a bumper to protect the toes.
- › Ankle strap seats foot firmly in shoe to reduce heel slippage and eliminate friction.
- › Removable forefoot closure provides security while eliminating buckle pressure.
- › Zoned outsole has an aggressive tread under the midfoot where traction is needed most.

Suggested Code: L3260

Compatible with:

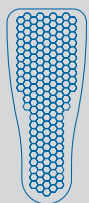

PQ PegAssist™

HeelWedge™

Size:	Women:	Men:	Product Codes:
XS	4 - 7	-	HQ0B
S	7.5 - 10	6 - 8	HQ1B
M	10.5 - 13	8.5 - 10	HQ2B
L	13.5+	10.5 - 12	HQ3B
XL	-	12.5 - 14	HQ4B



Video demonstration:

How to walk properly while wearing the HeelWedge™


Scan QR code or visit:

darcointernational.com/heelwedge

OrthoWedge™

Off-loading Shoe

Standard Surgical Shoe

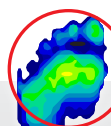

DARCO OrthoWedge®




The OrthoWedge™ is clinically proven to reduce weight bearing pressure on the forefoot which promotes faster healing when forefoot wounds or ulcerations are present.

- › Reduces forefoot pressure by as much as 57%!
- › Square-toe design acts as a bumper to protect the toes when k-wires are present and provides better left/right fit.
- › Ankle strap seats foot firmly in shoe to reduce heel slippage and eliminate friction.
- › Removable forefoot closure provides security while eliminating buckle pressure.
- › Zoned outsole has an aggressive tread under the midfoot where traction is needed most.

Suggested Code: L3260

Compatible with:

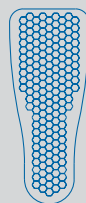

PQ PegAssist™



Body Armor® Toe Guard -
Square-toe

OrthoWedge™

Size:	Women:	Men:	Product Codes:
XS	4 - 7	-	OQ0B
S	7.5 - 10	6 - 8	OQ1B
M	10.5 - 13	8.5 - 10	OQ2B
L	13.5+	10.5 - 12	OQ3B
XL	-	12.5 - 14	OQ4B



Video demonstration:


How to walk properly while wearing the OrthoWedge™

Scan QR code or visit:

darcointernational.com/orthowedge

MedSurg™ with PegAssist™ (Sold separately)

Postoperative Shoe with Off-loading Insole

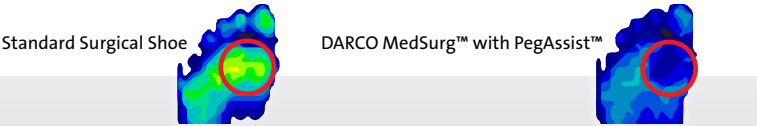

When used with the PTQ Series PegAssist™ off-loading insole, the MedSurg™ Shoe is one of the most effective wound care shoes on the market today. The relaxed rocker sole allows for easy ambulation through the gait pattern, eliminating high pressure strikes where wounds are most likely to occur.

- › Square-toe design acts as a bumper to protect the toes when k-wires are present and provides better left/right fit.
- › MetaShank™ protection provides more rigid control under the metatarsal heads.
- › Forefoot closure eliminates buckle pressure on the forefoot and can expand to accommodate dressing up to 17 inches in circumference.


MedSurg™

Suggested Code: L3260

PegAssist™

Suggested Code: A9283

Compatible with:

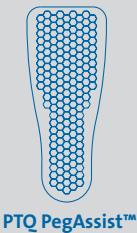

Body Armor® Toe Guard - Square-toe



MedSurg™

Size:	Product Codes	
	Women:	Men:
S	4 - 6	6 - 8
M	6.5 - 8	8.5 - 10
L	8.5 - 10	10.5 - 12
XL	-	12.5 - 14

PTQ Series PegAssist™ (Fits MedSurg™)

Size:	Product Codes	
	Women:	Men:
S	4 - 6	6 - 8
M	6.5 - 8	8.5 - 10
L	8.5 - 10	10.5 - 12
XL	-	12.5 - 14

GentleStep™

Diabetic Shoe



The GentleStep™ Shoe is a Medicare approved extra-depth shoe designed specifically for the needs of certain patients with diabetes. Conforms to the Medicare Therapeutic Shoe Bill.

- › Molded outsole conceals extra depth and provides lower profile and enhanced style.
- › Light-weight LYCRA® to provide extra comfort for patients with forefoot deformities or lesions
- › Firm heel counter ensures stable and comfortable fit.
- › Machine washable means that the GentleStep™ will stay looking nice for a long time.
- › Available in twenty-two sizes and three widths.

Suggested Codes: A5500, L3216, L3221

GentleStep™

Women:	Men:	Medium (B & C)	Wide (D & E)	Extra-Wide (EE & EEE)
5.5	4	GS055MB	GS055WB	GS055XB
6	4.5	GS060MB	GS060WB	GS060XB
6.5	5	GS065MB	GS065WB	GS065XB
7	5.5	GS070MB	GS070WB	GS070XB
7.5	6	GS075MB	GS075WB	GS075XB
8	6.5	GS080MB	GS080WB	GS080XB
8.5	7	GS085MB	GS085WB	GS085XB
9	7.5	GS090MB	GS090WB	GS090XB
9.5	8	GS095MB	GS095WB	GS095XB
10	8.5	GS100MB	GS100WB	GS100XB
10.5	9	GS105MB	GS105WB	GS105XB
11	9.5	GS110MB	GS110WB	GS110XB
11.5	10	GS115MB	GS115WB	GS115XB
12	10.5	GS120MB	GS120WB	GS120XB
12.5	11	GS125MB	GS125WB	GS125XB
13	11.5	GS130MB	GS130WB	GS130XB
13.5	12	GS135MB	GS135WB	GS135XB



GentleStep™ (Cont.)

Women:	Men:	Medium (B & C)	Wide (D & E)	Extra-Wide (EE & EEE)
14	12.5	GS140MB	GS140WB	GS140XB
14.5	13	GS145MB	GS145WB	GS145XB
15	13.5	GS150MB	GS150WB	GS150XB
15.5	14	GS155MB	GS155WB	GS155XB
16	14.5	GS160MB	GS160WB	GS160XB



Plantar Fasciitis Products

Pneumatic Night Splint

Posterior Night Splint



The Pneumatic Night Splint features a fully-integrated inflation/deflation system that can be easily operated with one hand. The internal air bladder is adjustable so every wearer can find the degree of dorsiflexion that works best. The natural cotton terry liner absorbs moisture and ensures a cool, comfortable fit and a non-skid sole for minimal ambulation when deflated.

- › Integrated air system makes inflation and deflation a snap. Now with no external parts to misplace.
- › Internal air bladder allows for adjustments to suit any wearer.
- › Cotton terry cloth liner is soft, comfortable and absorbent.
- › Provides the stretch to the plantar fascia that is the key to relief.

Suggested Code: L4396



Pneumatic Night Splint

Size:	Women:	Men:	Product Codes:
S	up to 6.5	-	NS1B
M	7 - 10	5 - 8	NS2B
L	10.5 - 13	8.5 - 11	NS3B
XL	13.5+	11.5 - 14	NS4B

Body Armor® Night Splint

Dorsal Night Splint



The Body Armor® Night Splint is the only night splint that actively engages the windlass mechanism of the foot, resulting in a specific and sustained stretch to the plantar fascia, and a sustained low load stretch to the flexor tendons, Achilles tendon and calf muscles.

- › Loading the windlass mechanism naturally brings the foot/ankle to a near 90 degree angle thus eliminating the “lever effect” and the resulting pressure on the apex of the ankle present in other dorsal models.
- › Elastic connector ring allows for natural flexion of the foot and toes.
- › Low profile design allows for maximum user comfort and compliance.
- › One-size-fits-most means reduced inventory.

Suggested Code: L4396



Body Armor® Night Splint

Size:	Product Code:
Universal	BADS



Video demonstration:
Proper application of the Body Armor® Night Splint

Scan QR code or visit:
darcointernational.com/body-armor-night-splint



Available in Three Colors:

The DARCO DCS™ Plantar Fasciitis Sleeve is woven to be anatomically correct. Wave compression technology on the plantar surface of the foot provides a sustained stretch on the plantar fascia. The DCS™ gives relief from the discomfort associated with plantar fasciitis by promoting circulation, reducing edema and foot fatigue, and relieving morning foot pain.

- › Wave compression provides a consistent and sustained stretch to the plantar fascia.
- › The DCS™ improves circulation and reduces edema.
- › 24 hour support - wear while sleeping or under socks when active.
- › Anatomically woven for maximum comfort.
- › Constructed of medical grade fabric that is latex free.
- › Comfort Zone on the Anterior Ankle Fold to prevent pinching or binding.
- › Low Profile Design.

Night Splint Wedge

Posterior Night Splint




The Night Splint Wedge is designed to alleviate heel pain associated with plantar fasciitis and achilles tendonitis. It features two 1/2 inch EVA wedges, each with 3° of inclination, that can be removed or added to adjust the desired dorsiflexion of the foot. A large supporting surface provides enough space for optimal comfort.

- › Removable wedges are included to create varying degrees of angle within the foot bed.
- › Fast and easy to use with a hook and loop fastener.
- › Cotton terry liner is soft, comfortable and absorbent.
- › Can be worn on the right and left.
- › Provides a sustained stretch to the plantar fascia.

Suggested Code: L4396

Wedges can easily be inserted/removed via the convenient front hook and loop fastener flap. They can also be combined for a total of six degrees of inclination.

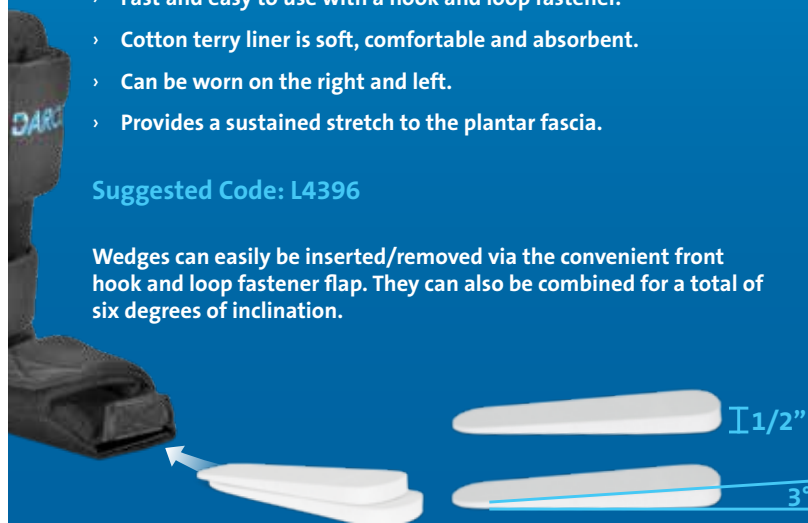

DCS™ - Plantar Fasciitis Sleeve (Limited U.S.A. availability)

Size:	Arch Circ.	Women:	Men:	Product Codes:		
				Blue:	Black:	Tan:
S	5 - 8 in	4 - 6.5	3 - 5.5	DCS-PF1	DCS-PF1-B	DCS-PF1-T
M	7 - 10 in	7 - 10.5	6 - 9.5	DCS-PF2	DCS-PF2-B	DCS-PF2-T
L	9 - 13 in	11+	10 - 13	DCS-PF3	DCS-PF3-B	DCS-PF3-T
XL	11 - 15 in	-	13.5+	DCS-PF4	DCS-PF4-B	DCS-PF4-T

Body Armor® Night Splint

Size:	Women:	Men:	Product Codes:
S	up to 6.5	-	NW1B
M	7 - 10	5 - 8	NW2B
L	10.5 - 13	8.5 - 11	NW3B
XL	13.5+	11.5 - 14	NW4B





Ankle Supports

Body Armor® Vario

Ankle Support



First line treatment for posterior tibial tendonitis and peroneal tendonitis. Customizable varus/valgus heel disks and arch height. Designed by a board certified foot and ankle specialist. Viable alternative to expensive custom ankle braces.

- › Heel posting disks allow for varus/valgus correction.
- › Customizable arch height supports medial midfoot.
- › 3/4 insole supports the arch and captures the calcaneus in comfort.
- › Trimmable straps can be adjusted to open/close in a specific position on the leg.

Suggested Code: L1906



Body Armor® Vario

Size:	Description	Product Codes:
Standard	Left	BAV2L
Standard	Right	BAV2R
Wide	Left	BAV3L
Wide	Right	BAV3R

Body Armor® Sport

Ankle Support



The Body Armor® Sport is an ankle joint orthosis which is applied during the acute phase of injury as well as during the process of rehabilitation.

- › Post-traumatic and preventive care.
- › Anatomically shaped stirrups ensures a perfect fit.
- › Unique strapping system prevents the forward shift of the talus protecting the anterior talo-fibular ligament.
- › Low profile uprights easily fits under sport or dress shoes.

Suggested Code: L1902



Body Armor® Sport

Left:	Women:	Men:	Product Codes:
S	5 - 8	< 7	BAS1L
M	9 - 12	7.5 - 10.5	BAS2L
L	13+	11 - 14	BAS3L
Right:	Women:	Men:	Product Codes:
S	5 - 8	< 7	BAS1R
M	9 - 12	7.5 - 10.5	BAS2R
L	13+	11 - 14	BAS3R

Web™ Ankle Brace

Ankle Support



The Web™ Ankle Brace is a true convertible ankle brace that provides unparalleled compression and can go from an acute brace to a sports support with a simple adjustment - no need to buy two braces.

- › Bungee closure provides superior compression and allows for adjustments on the run.
- › Removable medial and lateral stays are rigid enough to provide controlled restriction of inversion/eversion.
- › Multiple strapping configurations to further control inversion/eversion and to direct compression where needed.
- › Durable ballistic nylon shell is breathable and conforms to the foot for a comfortable fit.
- › High tech spacer material padding wicks moisture away from foot.
- › Universal fit allows use on right or left ankle.

Suggested Code: L1902

Web™ Ankle Brace


Size:	Women:	Men:	Product Codes:
XS	4 - 6	-	WB0
S	6.5 - 9	6 - 7	WB1
M	9.5 - 11	7.5 - 10	WB2
L	11.5 - 13	10.5 - 12	WB3
XL	13.5 - 15	12.5 - 14	WB4



Video demonstration:
Proper application of the Web™ Ankle Brace

Scan QR code or visit:
darcointernational.com/web-ankle



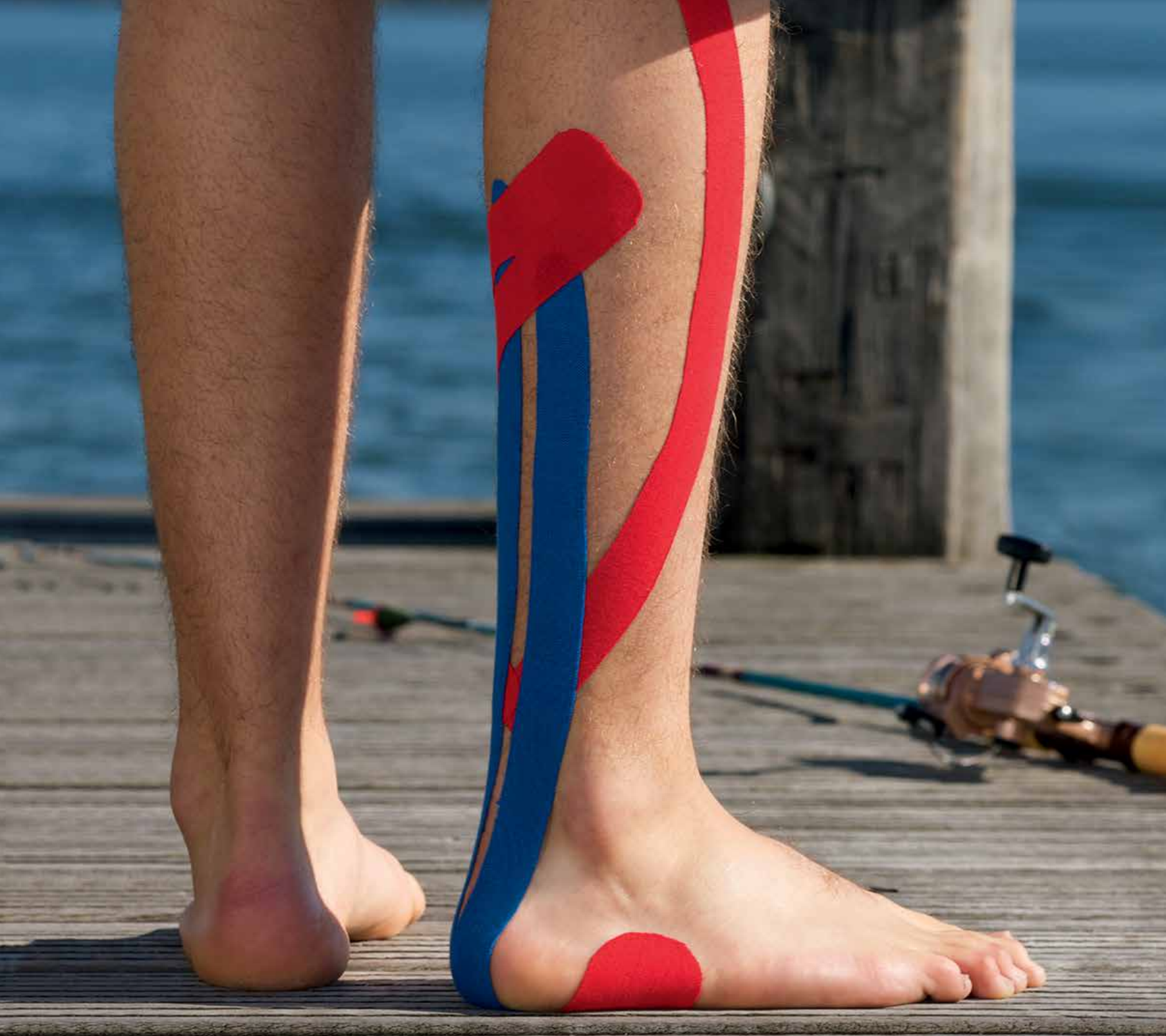

Kinesiology tape

MECRON Elastic Tape

Kinesiology Tape



MECRON Elastic Tape is an elastic sport and therapy tape made especially for use in applied kinesiology. Kinesiology tape is used worldwide by athletic trainers and physicians alike because it provides portable and immediate treatment of muscle, fascia and tendon symptoms and advances performance by way of continuous receptor stimulation.

- › Stimulates circulation and increases both blood flow and the flow of lymphatic fluids to promote healing of the affected area.
- › Improves performance by facilitating greater range of motion.
- › Water-resistant tape stays on for days or until the wearer wants to remove it.
- › Cost effective.

Available in Five Colors:



MECRON Elastic Tape

Color:	Length:	Width:	Product Code
Red	16.5 feet	2 inches	BAET-R
Blue	16.5 feet	2 inches	BAET-N
Yellow	16.5 feet	2 inches	BAET-Y
Black	16.5 feet	2 inches	BAET-B
Beige	16.5 feet	2 inches	BAET-T



Orthotic Insoles

ProMotion® Plus

Orthotic Insoles



The DARCO ProMotion® Plus orthotic insole features a full length PORON® Cushioning layer which provides greater comfort and doesn't bottom out long term.

- › Full length PORON® Cushioning foam layer provides superior shock absorption
- › High quality top layer fabric features anti-microbial odor control and low friction
- › Heel to toe cushioning foam that won't flatten or bottom out provides a superior feel and long term wear benefits
- › EVA covered polypropylene heel and arch support. Provides a firm support that cradles the heel and arch area. The EVA surface prevents the insole from shifting while walking and adds durability
- › Available through health care providers
- › Economically priced


Suggested Code: L3040

ProMotion™ Plus

Size:	Women:	Men:	Product Codes:
3	5 - 5.5	-	PMP03
4	6 - 6.5	-	PMP04
5	7 - 7.5	5 - 5.5	PMP05
6	8 - 8.5	6 - 6.5	PMP06
7	9 - 9.5	7 - 7.5	PMP07
8	10 - 10.5	8 - 8.5	PMP08
9	11 - 11.5	9 - 9.5	PMP09
10	12 - 12.5	10 - 10.5	PMP10
11	13 - 13.5	11 - 11.5	PMP11
12	-	12 - 13.5	PMP12
14	-	14 - 14.5	PMP14

poron | performance
cushioning

PORON is a licensed trademark of Rogers Corporation.

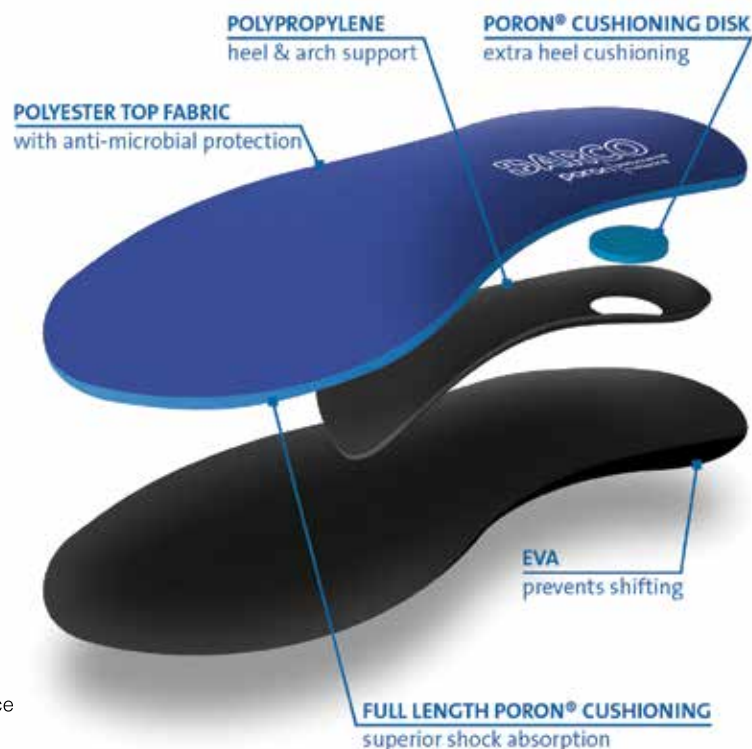



Pediatric Products

MedSurg™ (Pediatric)

Postoperative Shoe



The Pediatric MedSurg™ shoe is a versatile, comfortable and lightweight post-op shoe that is perfect for the smaller foot.

- › Square-toe design acts as a bumper to provide extra protection and facilitates better left/right fit.
- › MetaShank™ protection provides more rigid control under the metatarsal heads.
- › Strapless closure eliminates buckle pressure on the forefoot and can expand to accommodate large dressings.
- › Relaxed rocker sole reduces plantar pressure on the heel and forefoot by over 25%.

Suggested Code: L3260

Pediatric MedSurg™

Size:	Children:	Product Code:
Pediatric	12 - 1	MQPB

Original MedSurg™ (Pediatric)

Postoperative Shoe



The DARCO medical-surgical shoe is the shoe that started it all. Our original MedSurg™ shoe was invented over 30 years ago and has since become known as the industry standard. Often referred to generically as “the DARCO shoe”, it is anything but generic and its long list of features and excellent quality make it one of the best values in the business.

- › MetaShank™ provides superior control of forefoot motion to ensure postoperative healing remains undisturbed. Better protection after osteotomies.
- › Reinforced padded heel grips the foot and reduces slippage.
- › Breathable mesh upper light weight material allows for proper ventilation and conforms to bony abnormalities.
- › Extra long straps accommodate even the bulkiest dressings.

Suggested Code: L3260

Pediatric Original MedSurg™

Size:	Children:	Product Code:
Pediatric	12 - 1	MSPN

SlimLine™ (Pediatric)

Cast Boot



The Pediatric SlimLine™ cast boot can be adapted to fiberglass casts, bulky compression bandages and a variety of wound care applications, making it one of the most versatile cast shoes on the market.

- › Upper ankle strap adds stability.
- › Multiple sizes to assure correct fit.

Suggested Code: L3260



Pediatric SlimLine™ Cast Boot

Size:	Children:	Product Codes:
Pediatric Small	5 - 7	SLO1NP
Pediatric Med	8 - 11.5	SLO2NP
Pediatric Large	12 - 1.5	SLO3NP

CBO™

Open Toe Cast Boot



The Darco CBO™ is a low profile multi use cast boot intended to protect casts of all types and heavy compression bandages.

- › Heavy duty canvas upper that protects the cast from the elements.
- › Multi layer EVA sole provides extra cushioning for ease of ambulation and durability.
- › The rocker sole helps ease pressure when ambulating and allows an easy roll through the gait pattern.
- › Durable hook and loop fastener straps seat and secure the casted foot and keep the boot snug around the cast/bandaging.
- › The CBO™ is lightweight and low profile.

Suggested Code: L3260



CBO™

Size:	Children:	Product Code:
Pediatric	12 - 1	CBOP

Diagnostic Products

FootFilament™

Sensitivity Tester




The compact FootFilament™ opens to reveal a 10g monofilament. The raised dots on the front and back indicate test sites.

- › Assess touch and pressure sensation in large nerve fibers.
- › Compact and easy to use.

Monofilament Tester

Single-Use Sensitivity Tester

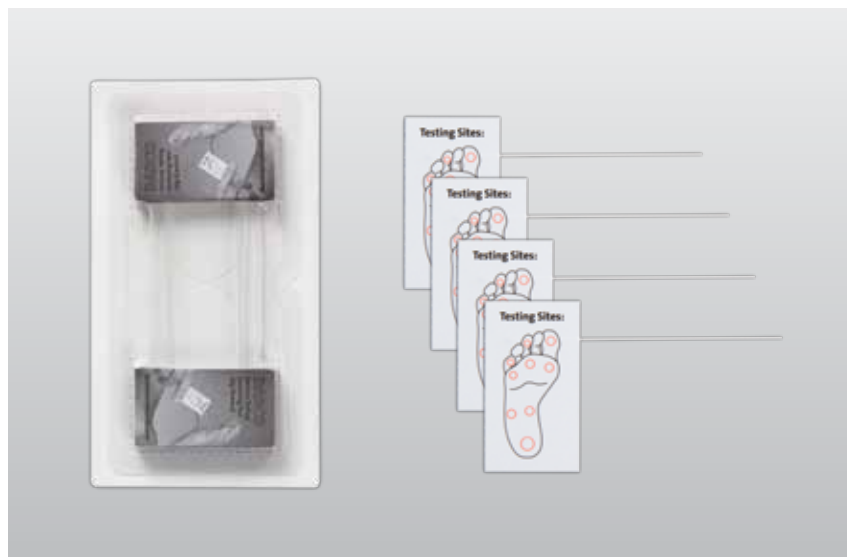

The single use monofilament tester is an excellent tool to determine nerve issues or neuropathy in patients.

- › Assess touch and pressure sensation in large nerve fibers.
- › Single use, 10g calibrated monofilament.
- › Economical 20 per pack.
- › Touch points on tester handle.


FootFilament™

Product Code:

FF

Testing Sites:

Plantar surface of the first, third, and fifth toes

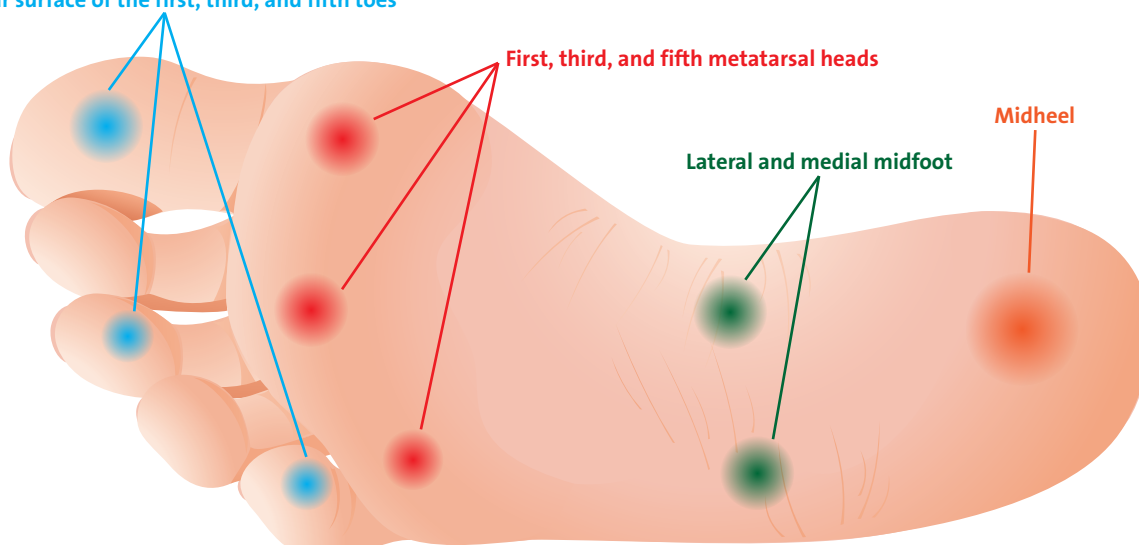

Monofilament


Product Code:

MFT


Insole Sizes and Measurements

Measurements in Inches


OrthoWedge™, HeelWedge™, APB™, SlimLine™

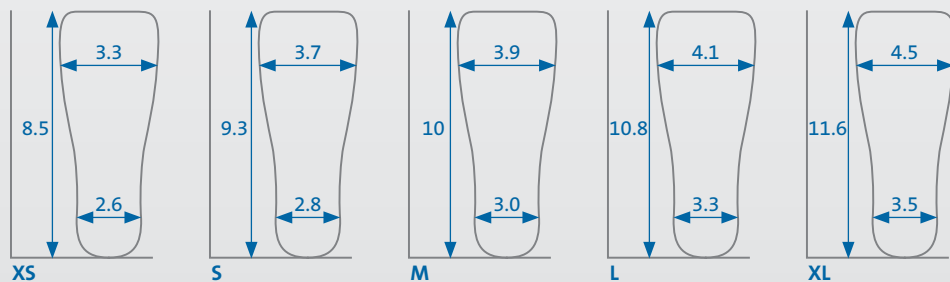


Square Toe MedSurg™

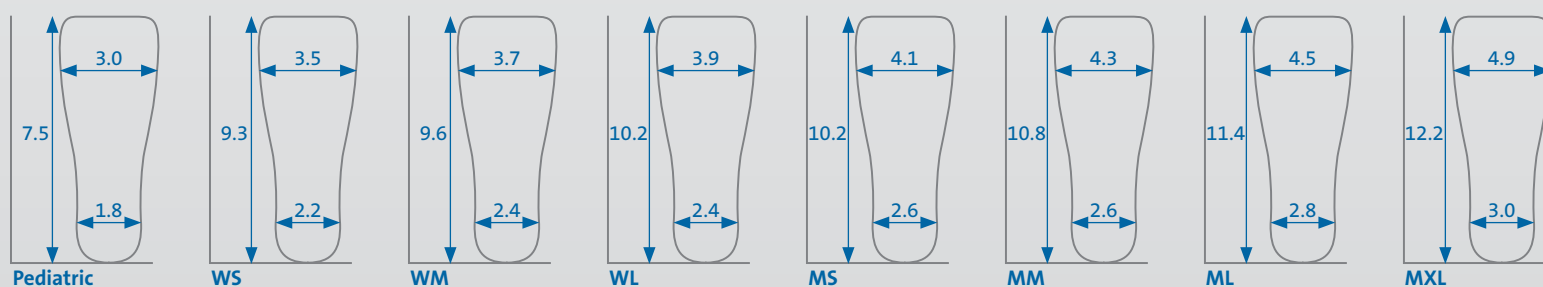


AirTraveler™ Walker

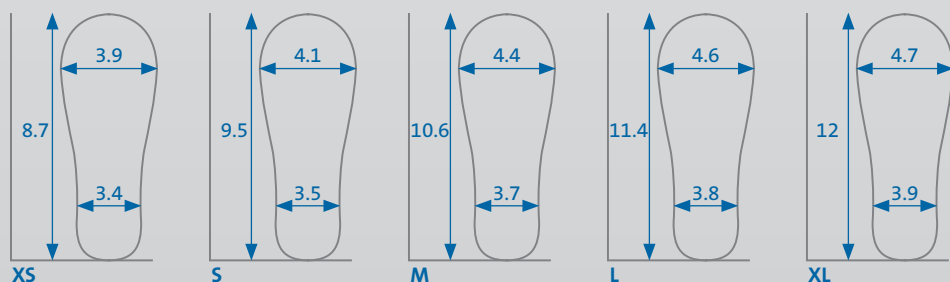


Body Armor® Walker II

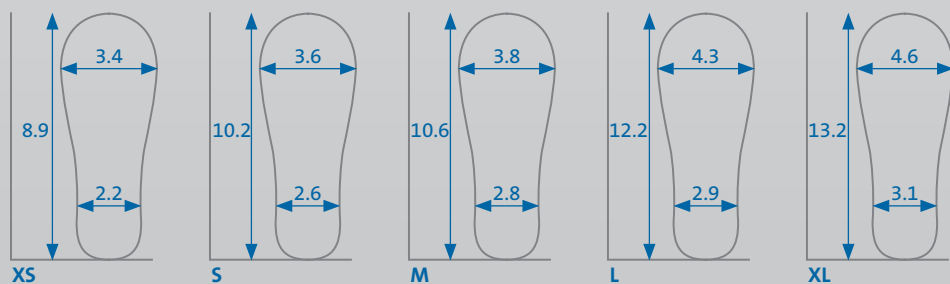


WCS™ Wound Care Shoe System

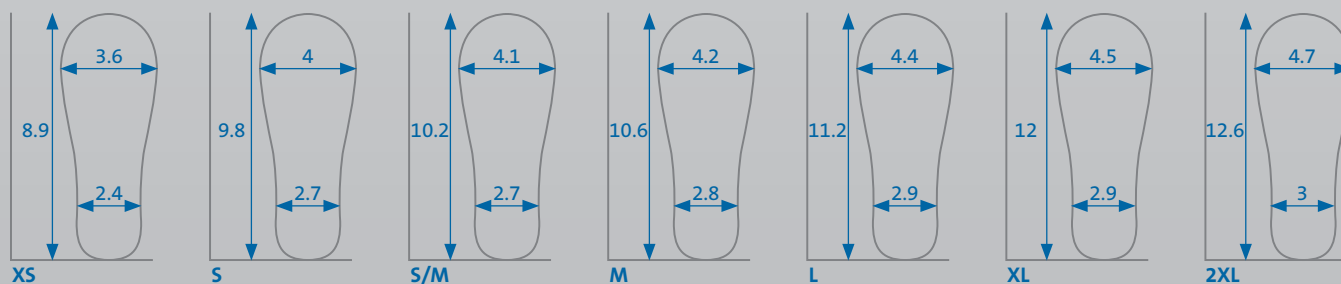


Original MedSurg™

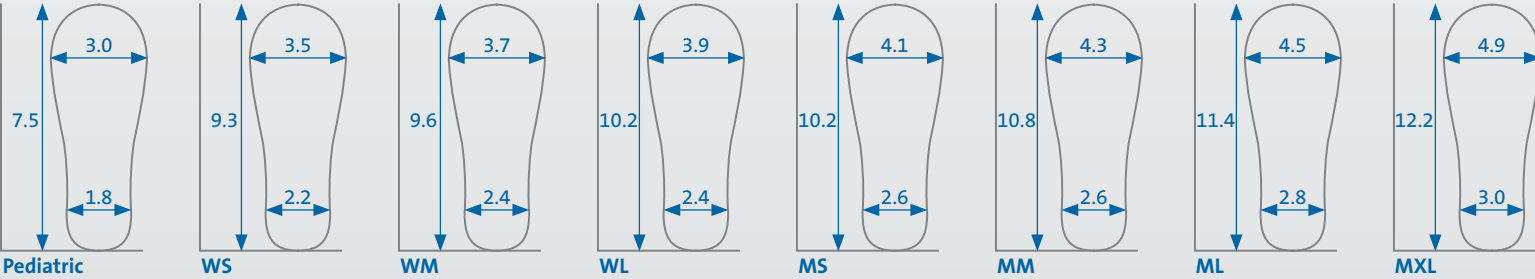


FX Pro™ Walker

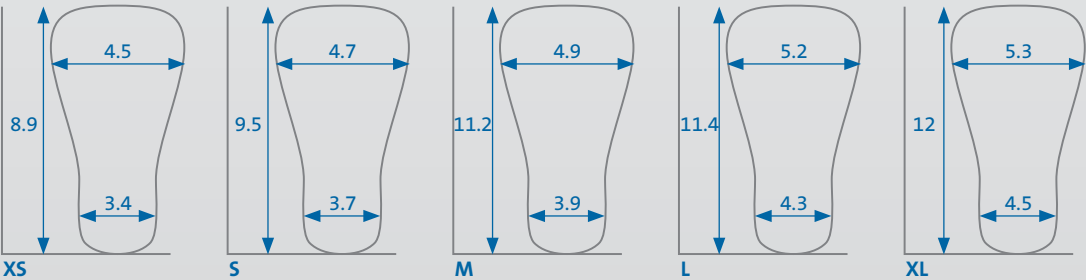

Body Armor® Cast Shoe

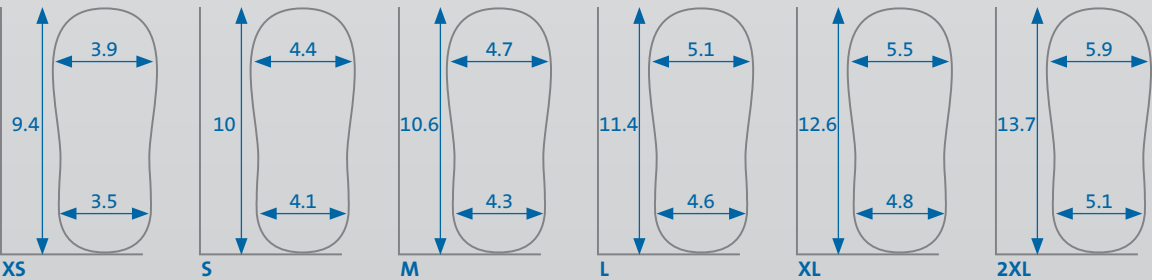

Diabetic Treatment Product Guide

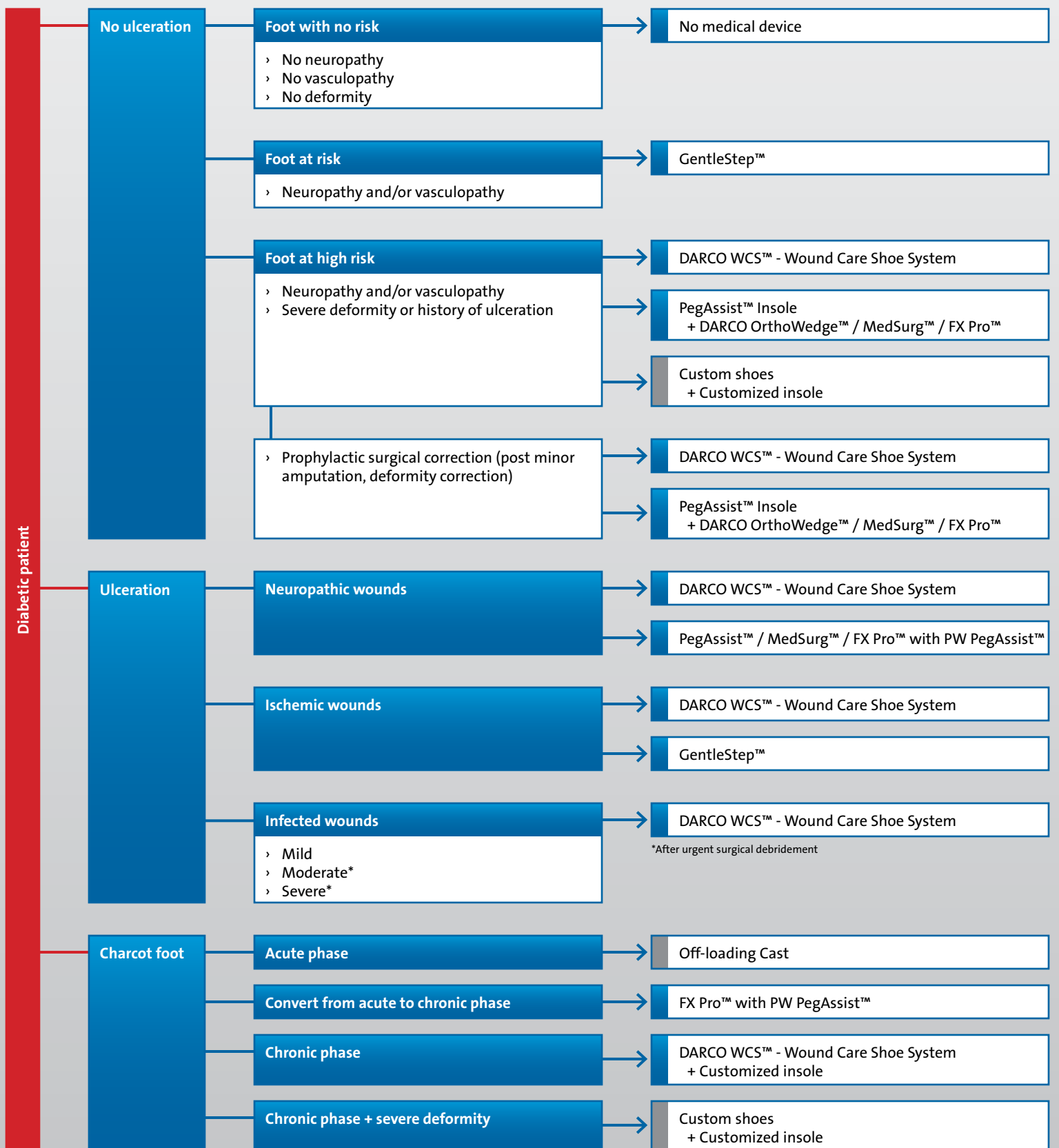

Recommended product by DARCO

Recommended method

Product Videos



Video demonstration:

How to walk properly while wearing the HeelWedge™

Scan QR code or visit:

darcointernational.com/heelwedge



Video demonstration:

How to walk properly while wearing the OrthoWedge™

Scan QR code or visit:

darcointernational.com/orthowedge



Video demonstration:

How to properly apply the TAS™

Scan QR code or visit:

darcointernational.com/tas



Video demonstration:

Proper application of the Body Armor® Night Splint

Scan QR code or visit:

darcointernational.com/body-armor-night-splint



Video demonstration:

How to properly modify and apply the WCS™

Scan QR code or visit:

darcointernational.com/wound-care-shoe-system



Video demonstration:

Proper assembly and application of the PegAssist™ Off-loading Insole

Scan QR code or visit:

darcointernational.com/pegassist



Video demonstration:

Proper application of the Web™ Ankle Brace

Scan QR code or visit:

darcointernational.com/web-ankle



PORON is a licensed trademark of Rogers Corporation.
Plastazote® is a registered trademark of Zotefoams PLC.
LYCRA® is a registered trademark of INVISTA

**Publisher and responsible for
content**

DARCO International
810 Memorial Blvd.
Huntington, WV 25701 | USA
Toll Free: 800-999-8866
Toll Free Fax: 877-201-7082
Voice: 304-522-4883
Fax: 304-522-0037

Last update

© DARCO | October 2017

Product photography

DARCO International
DARCO (Europe) GmbH



www.darcointernational.com

DARCO